


You Don't
Need
A Laser!

VascuTouch

for the quick and easy removal of:

- Telangiectasia (couperose)
- Spider Veins
- Cherry Angiomas
- Skin Tags
- Vascular Blemishes

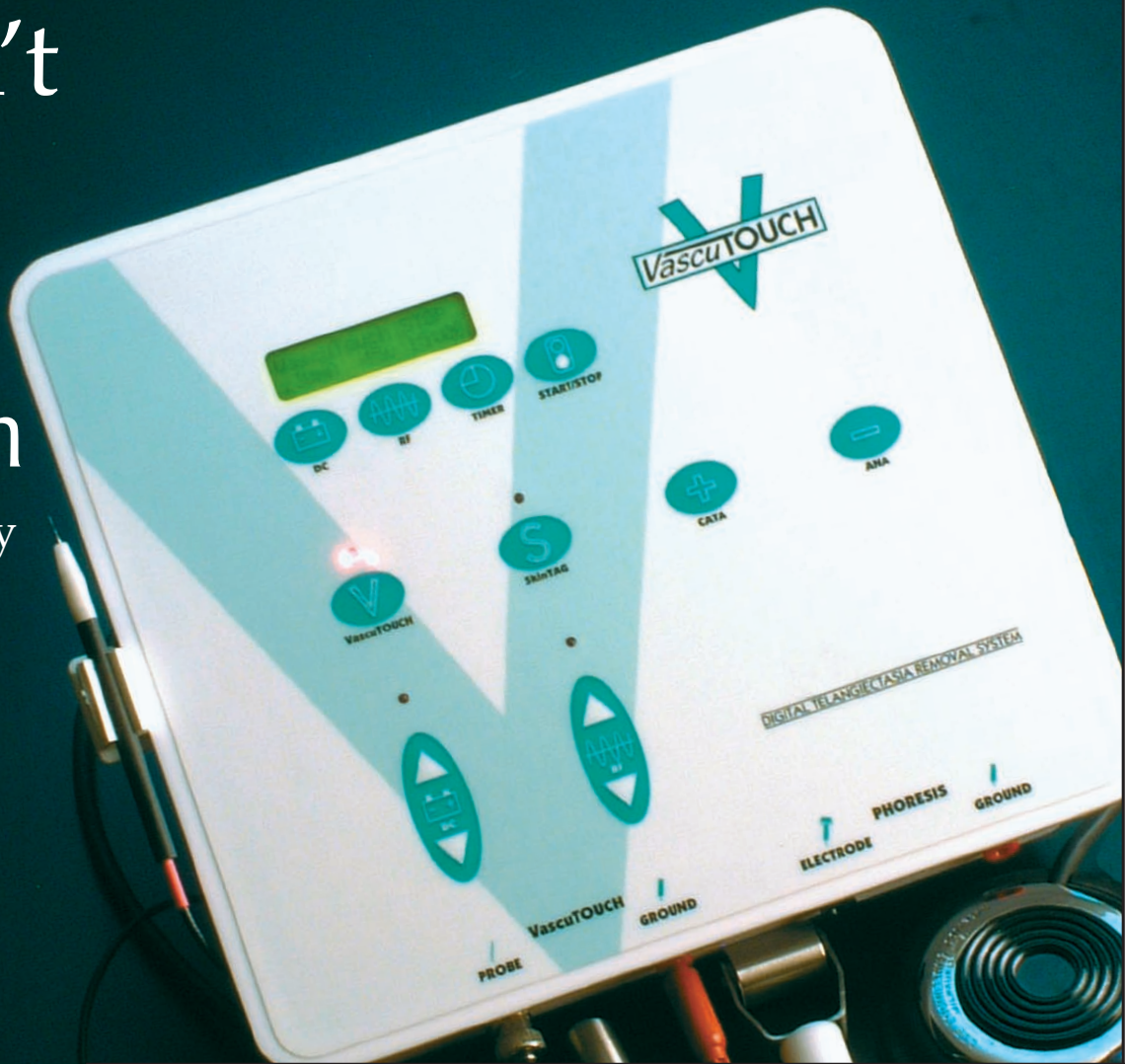

BEFORE



AFTER



One Foot Pedal

Four Function

- Radio Frequency RF
- Galvanic
- Phoresis
- Air – Cools/Numbs

VascuTouch is a quick and effective treatment for the removal of unsightly facial veins and skin tags.

THE VASCUTOUCH PROVIDES AN INEXPENSIVE means to quickly and easily remove vascular lesions such as Telangiectasia (distended capillaries), cherry angiomas, spider veins, and skin tags found on the face and body. The VascuTouch yields superior results when compared to other electronic devices that commonly cause scarring and hypo and hyperpigmentation. When compared to laser treatment, the VascuTouch produces similar results without the pain, bruising, or high treatment costs. The VascuTouch is an indispensable tool for treating cosmetic vascular problems.



Easy to learn procedure removes telangiectasia, cherry angiomas, spider veins and skin tags.

THE VASCUTOUCH IS A BETTER WAY TO REMOVE TELANGIECTASIA. This quick and simple treatment will not cause scarring or hyperpigmentation. VascuTouch uses very low levels of Radio Frequency (RF) and DC current simultaneously to electrocoagulate the vessel, without desiccating surrounding tissue. This treatment is self-limiting and vessel specific, resulting in minimal skin reaction. Low treatment energy does not adversely affect the dermis to cause scarring or any other major reaction.

THE VASCUTOUCH OFFERS A PHORESIS MODE TO:

- ✓ calm the skin and reduce swelling;
- ✓ increase penetration of "ampule" and other skin care products;
- ✓ facilitate disen crustation treatments

SYSTEM INCLUDES:

- VascuTouch Unit
- 30 Sterile disposable teleprobes
- 30 Sterile disposable skin tag probes
- VascuTouch Users Manual

NOTE: Each state has a regulatory authority responsible for governing the use of electrical probe devices by a nonmedical therapist. You should inquire of your state regulatory agency as to any determination regarding treatment with this type device.

HOW LONG DOES A VASCUTOUCH TREATMENT TAKE? Most blemishes are removed in seconds. Depending on the number of capillaries, most conditions can be successfully and permanently corrected in one or two 10–15 minute sessions.

IS SKIN PENETRATION BY A NEEDLE NECESSARY? No! A **non-invasive** sterile probe is placed on the skin surface above the vessel to be treated. There is no pressure or penetration of the skin during treatment.

HOW IS THE NON-INVASIVE PROCEDURE DONE? The surface of the skin is cleansed and prepped. A fine tipped sterile probe is placed on the surface of the skin. Low-level RF and DC current are passed through the probe to the vessel. The tiny vessel is clotted or coagulated and blood stops flowing. With proper post-treatment care the vessel is naturally absorbed into the surrounding tissue. Typically within three to four weeks the vessel disappears.



Telangiectasia or Couperose



Cherry Angioma

Introducing RetroCaine

a topical anesthetic that works in less than 10 minutes to minimize patient discomfort.

(RetroCaine contains lidocaine and aloe vera.)
Order your sample today!

OPTIONAL AIR DESENSITIZATION MODE.

The VascuTouch offers an optional function that creates a desensitizing effect on the skin. Directing a stream of blowing air at the treatment site fools nerves in the skin, creating an anesthetic effect that minimizes any patient discomfort.